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The HARMONY™ Cushion is manufactured in the U.S.A. by The ROHO Group, Inc., 100 N. Florida Ave., Belleville, IL 62221-5429 U.S.A.

© 2003 The ROHO Group

The following are trademarks and registered trademarks of The ROHO Group, Inc.: ROHO®, AIR FLOATATION™, DRY FLOATATION® and HARMONY™.

## HARMONY™ Cushion

### About this Manual

The HARMONY Cushion User Manual includes information regarding the use and proper care of your new cushion. Please read these instructions in full before using this product. Proper adjustment of the AIR FLOATATION™ cushion overlay is critical to assist in protecting your skin from breakdown.

### About the HARMONY Cushion

The HARMONY Cushion combines an AIR FLOATATION Cushion Overlay and a contoured foam base. By combining the characteristics of the original ROHO DRY FLOATATION® Technology with a contoured foam base, the HARMONY Cushion enhances skin protection, positioning, mobility, and comfort. Tissue protection focuses on your ischia, sacrum, and coccyx, while stability and positioning are targeted at the pelvis and thighs. An easily accessible valve allows you to adjust the cushion insert, and a fluid-resistant, antimicrobial cover combines fluid protection with breathability.

### CAUTION

To avoid the hazards identified below, check your cushion daily to determine that the cushion is properly adjusted and in proper working order.

**DEFLATION:** Loss of air in cushion can result in bottoming out (see below), eliminating (if not immediately fixed) the characteristics of the cushion that assist in healing and preventing ischemic ulcers. It is your responsibility to check the cushion daily or have it checked and to use the cushion appropriately.

**BOTTOMING OUT:** Too much air being released from your cushion can cause your lowest bony prominence to touch the cushion base. This decreases the therapeutic value of the cushion and could result in skin breakdown (ischemic ulcers). Product should be adjusted so approximately 1/2" - 1" of air is maintained between the AIR FLOATATION cushion overlay and your lowest bony prominence.


#### DO NOT:

- expose cushion to pins or sharp objects.
- expose cushion or cover to hot ashes or any open flames.
- use the air valve as a handle.
- use oil based lotion or lanolin on your cushion.
- use the cushion as a water flotation device.
- autoclave cushion because closed valve may increase internal pressure and cause cushion to rupture.

Please contact The ROHO Group or your nearest Rehabilitation Technology Supplier or distributor immediately if deflation or bottoming out that you cannot remedy is occurring for any reason.

## HARMONY Cushion Parts Detail


---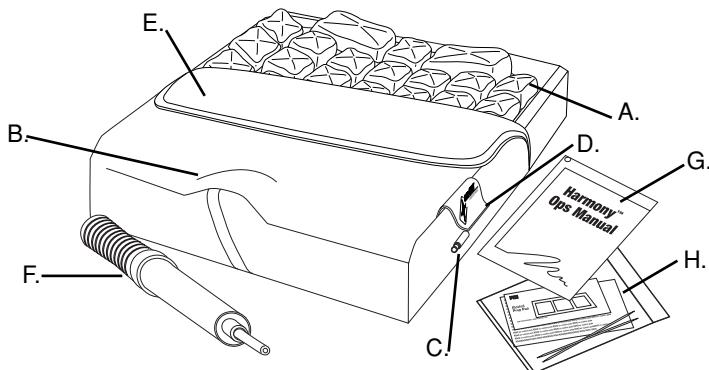

- A. Cushion Overlay / Cells:** The ROHO Group refers to its individual air pockets as "cells". The section of the cushion with the air cells is referred to as the cushion overlay, or AIR FLOATATION cushion overlay.
- B. Contoured Foam:** A contoured foam base that is specially shaped to enhance stability and position pelvis and thighs. The foam base sits in the bottom pocket of the cover.
- C. Air Valve:** The valve is used to adjust the amount of air in the cushion overlay. The valve should NOT be used for lifting or carrying.
- D. Valve Flap:** The valve is hidden by a flap on the left side of the cover
- E. Fluid-resistant, anti-microbial, stretchable cover:** A cover is included to protect the foam from outside fluids. It is also anti-microbial and stretchable. The Cushion overlay sits in the top pocket of the cover.
- F. ROHO Pump:** The ROHO Pump is used to inflate the AIR FLOATATION cushion overlay.
- G. User Manual:** Each ROHO Cushion includes a User Manual that provides instruction on adjustment, care, repairs, and warranty.
- H. Patch Kit:** A Patch Kit is included to repair small holes.

## **Placing the HARMONY Cushion on Your Wheelchair**

Place the HARMONY Cushion on your wheelchair so that the contoured foam area is in the front portion of the wheelchair seat. The AIR FLOATATION cushion overlay sits in the upper portion of the cover. The cushion should be placed on the seat so that the cushion overlay cells are facing upward.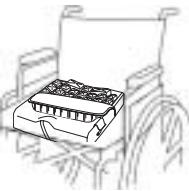


The cover should be on the cushion so that the zipper area is in the back portion of the wheelchair seat and the side logo is in the front left corner. The non-skid material is on the bottom of the cover.

## **Adjusting the AIR FLOATATION Cushion Overlay**

**IMPORTANT! Do Not Skip This Section.**

**Step 1:** Open air valve by twisting it counter-clockwise. (Air valve is located behind flap on the left side of the cushion, toward the front.)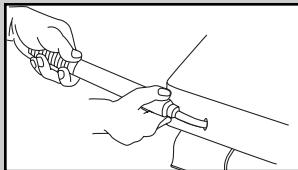


**Step 2:** Attach the rubber hose on the ROHO Pump to the end of the valve. Inflate the cushion overlay with the pump until the cushion begins to arch.


**Step 3:** Remove the pump hose from the valve and close the valve by twisting it clockwise.

**Step 4:** Sit on the HARMONY Cushion in your normal sitting position with armrests and footrests properly adjusted.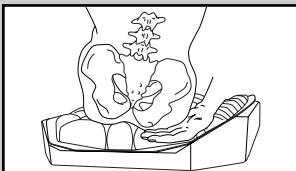

**Step 5:** Slide your hand beneath your buttocks to locate your lowest bony prominence, usually the ischial tuberosities. If necessary, your clinician or caregiver should assist you.

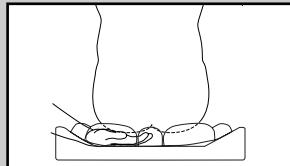**Step 6:** Leaving one hand in place beneath the lowest bony prominence, use the other hand to open the valve. As air is released from the cushion overlay, you will begin to sink into it. When your fingertips touch its base, close the valve. Approximately 1/2" - 1" of air should be between your lowest prominence and the base of the cushion.

### **IMPORTANT Adjustment Notes**

- You, your clinician(s) and / or caregiver(s) should check the cushion overlay **daily** to ensure that you have not bottomed out (ie. lost air and sitting on the base of the cushion). If you bottom out, the therapeutic value of the cushion overlay is lost. You should also check to be sure the cushion overlay is not over-filled with air. The cushion overlay will work best with approximately 1/2" - 1" of air between you and its base.
- The AIR FLOATATION cushion overlay may be affected by changes in altitude. You may have to repeat Steps 1 - 6 above as you change altitudes.

## **Cleaning Your HARMONY Cushion**

**To clean the AIR FLOATATION cushion overlay:** gently pull overlay out of the cover and the valve out of the side of the cover. Deflate the cushion, close valve(s) and place in a large sink. Gently scrub the air cells and the spaces between them with a soft plastic bristle brush, sponge, or washcloth. The ROHO Group suggests using a mild soap and warm water to clean the overlay. Do not use bleach when cleaning this product.

**To clean the foam base:** wipe gently with a cloth dampened with water and/or a household cleaning disinfectant. Allow foam to dry thoroughly. Do not immerse the foam in water.

**To disinfect the foam base:** wipe gently with a cloth dampened with a household cleaning disinfectant. Allow foam to dry thoroughly. Do not immerse the foam in water. **Do not autoclave.**

**To clean the cover:** Machine wash in cold water. No bleach. Air dry.

## **REPAIRING LEAKS**

You should protect your HARMONY Cushion from pins or other sharp objects and from exposure to hot ashes or any open flame. If your AIR FLOATATION cushion overlay is punctured, you can repair it using the Patch Kit included. Follow the directions with the kit to repair the leak. The patch should be monitored periodically to ensure its effectiveness.

## **RETURN POLICY**

All returns require prior authorization from The ROHO Group and are subject to a restocking charge. If you live outside the U.S.A., contact the distributor or dealer in your country or contact ROHO International to find a distributor near you.

## **CONTACT INFORMATION**

If you have any questions concerning repairs or the proper care and use of your ROHO product, call TOLL-FREE 1-800-851-3449 in the U.S.A. and Canada or outside the U.S.A. call 1-618-277-9150. Or, visit our website at [www.therohogroup.com](http://www.therohogroup.com). Send cushions to The ROHO Group, 100 N. Florida Ave., Belleville, IL 62221-5429 U.S.A.

## The ROHO Group 18-Month Limited Warranty

### **What is covered?**

The ROHO Group covers any defects in materials or workmanship of the HARMONY Cushion.

### **For how long?**

18 months from the date the HARMONY Cushion was originally purchased as a new cushion, no matter who is the present owner.

### **What we will do.**

Within a reasonable amount of time after you return the cushioning product to us, we will repair any defect in material and workmanship and ship it back to you, all free of charge (within the warranty period).

### **What we will not do.**

We will not repair a cushion free of charge if it has been misused or damaged by an accident. This also includes damages stemming from "acts of God", eg., flood, tornado, earthquake, or other catastrophic events, eg., fire.

### **What the customer must do.**

The customer must send the HARMONY Cushion to us with the original sales ticket or other evidence of date of purchase, and he/she must pay all delivery costs. It must be sent to The ROHO Group, 100 N. Florida Ave., Belleville, IL 62221-5429 U.S.A. We will not require the original sales ticket if you or the previous owner, if applicable, have completed and returned the warranty registration card, attached to this brochure, and we have it on file. Failure to submit the original sales ticket or failure to have the warranty registration card in our file voids this warranty.

### **Any other questions.**

Call us toll free in the U.S. or Canada at 1-800-851-3449, or outside the U.S. and Canada at 1-618-277-9150.

### **Disclaimer**

Any implied warranties, including the MERCHANTABILITY and FITNESS FOR A PARTICULAR PURPOSE are also limited to 18 months from the date of original purchase and any action for a breach of such warranties or any express warranties herein must be commenced within this time.

### **Your State Law Rights and the Effect of this Warranty Upon Them**

This warranty restricts the duration of all implied warranties to 18 months and limits or excludes recovery for incidental or consequential damages. Some states do not allow limitations on how long implied warranties will last and some states do not allow the exclusion or limitation of consequential or incidental damages, so those provisions may not apply to you. While this warranty give you specific legal rights, you may also have other rights which vary from state to state.

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**the ROHO group**  
shape fitting technology®

**REGISTRATION CARD**  
**18 Month Limited Warranty**  
PLEASE COMPLETE, DETACH AND  
MAIL FOR WARRANTY REGISTRATION

Name \_\_\_\_\_

Street No. \_\_\_\_\_

City, State \_\_\_\_\_

Zip & Country \_\_\_\_\_

Area Code & Telephone (      ) \_\_\_\_\_

User's Age?  18 and under  19-34  35-54  55-65  Over 65

**PRODUCT INFORMATION**

Purchased From \_\_\_\_\_

Date Purchased \_\_\_\_\_

Is this your first ROHO cushion?  Yes  No

Serial Number of cushion \_\_\_\_\_  
(Printed on back of cushion along lip)

Type of Product:  QUADTRO® SELECT®  ENHANCER®  
 HIGH PROFILE®  HIGH PROFILE® DUAL  
 LOW PROFILE®  LOW PROFILE® DUAL  
 HARMONY™  Other \_\_\_\_\_  
 nexus® SPIRIT® (Please specify)

1. How did you learn about  
DRY FLOATATION® cushions?

- therapist  friend  
 doctor  advertisement  
 dealer

3. How was your ROHO® cushion  
paid for?

- insurance  Medicare  
 self/family  
 Other \_\_\_\_\_  
(Please specify)

2. Why the ROHO system?

- prescribed  
 suggested by a ROHO product user  
 unsatisfied with other brands

4. How many cushions do  
you currently own?


- 1  3  more than 4  
 2  4

By signing this card, you agree that The ROHO  
Group may contact you in the future with  
additional product or promotional support.



\_\_\_\_\_

signature

LITG01005 11/04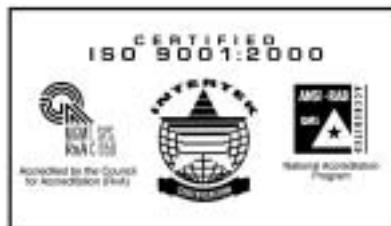

NO POSTAGE  
NECESSARY  
IF MAILED  
IN THE  
UNITED STATES

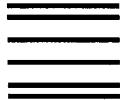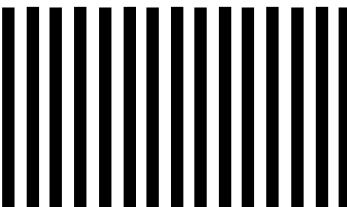## BUSINESS REPLY MAIL

FIRST CLASS PERMIT NO. 846 BELLEVILLE, ILLINOIS, U.S.A.

POSTAGE WILL BE PAID BY ADDRESSEE

100 N. Florida Avenue  
Belleville, Illinois 62221-5429 U.S.A.